

# LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

*The Warranty shall not apply under the following conditions:*

problems arising from normal wear; problems arising from failure to adhere to the product instructions; problems arising from misuse, negligence, accident or improper operation, maintenance or storage; problems arising from modifications or unauthorized repairs, parts or attachments; products where the serial number has been removed or defaced; Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever.

Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.



## Home Bed Assist Handle



**⚠ WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)



**SYMBOL GLOSSARY**  
For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

R180129


Manufactured for: Dynarex Corporation  
10 Glenshaw Street • Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)

Made in China

# INSTRUCTIONS


1. Remove mattress from bed.
2. Attach the rail to the base support frame by depressing the brass lock pins and sliding the rail into the tubing until the lock pins “snap” through the positioning holes. (Figure 1).

FIGURE 1

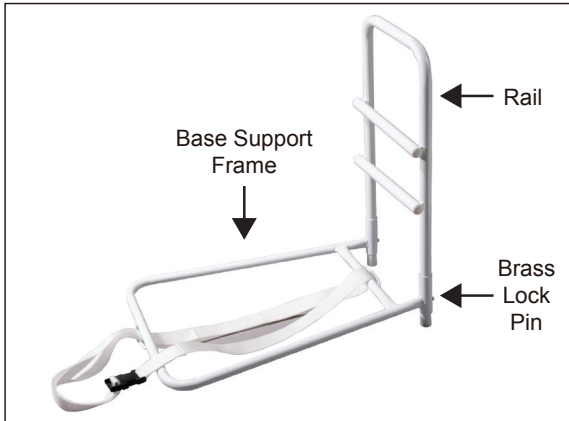

3. Place assembled base support frame on top of the bed so that the rail edge is on the outside of the frame of the bed.
4. Unbuckle the strap and lay flat so that the strap is exposed on both sides of the bed (Figure 2).

FIGURE 2

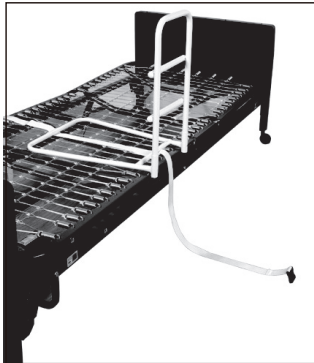

5. Place mattress on bed.
6. Buckle the strap around the box spring or mattress and adjust by tightening the loose end of the strap (Figure 3).

FIGURE 3



## WARNING

- This assist handle is intended for assisting in and out of bed only. **DO NOT** use for other purposes.
- **DO NOT** exert side pressure on the Home Bed Assist Handle; this could deform or break it.
- **DO NOT** use the Home Bed Assist Handle as a push handle for moving the bed; this could deform or break it.